

**Document Revision History:**

Rev. #	Section Effected	Rev. Details	Rev. Date
00	Nil	Document Developed First Time	01 July13

	Name	Signature	Date
Approved by:	Abdul Rehman Chief Executive		01 July 2016

# Instructions For Use (IFU)

**Non-Sterile, Reusable;  
Electrosurgical Coated Surgical Instruments**

Doc. # TF-06/06  
Rev. # 00  
Date: 01 July 2016



Group	Division	User	Document Control Stamp
RB	Electrosurgical	QA	
<b>Warning:</b> No part of this document may be reproduced, stored in a retrieval system, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording in any language or otherwise, without the prior written permission of Management.			

# INSTRUCTION FOR USE (IFU)

## Non-Sterile, Reusable; Coated Electrosurgical Instruments

Doc. # TF-06/06  
Rev. # 00  
01 July 2016  
Page 2 of 3

### INSTRUCTIONS FOR USE & CARE REUSABLE ELECTROSURGICAL COATED INSTRUMENTS

This product is reusable and is supplied **NON-STERILE**. Process the coated instruments through cleaning and sterilization prior to initial use, following guidance as outlined in this IFU. For questions or additional information on our complete line of coated electrosurgical instruments, please contact Anthony Products at 800-428-1610 or online at [www.anthonypproducts.com](http://www.anthonypproducts.com).

**INDICATIONS:** Reusable coated instruments are designed to be used in soft tissue surgical procedures during electrosurgical procedures.

**WARNING:** Any use of these instruments for tasks other than for which it is indicated will usually result in a damaged or breakage.

**CAUTION:** Federal (USA) law restricts this device to sale by or on the order of a physician.

**EXTENDING INSTRUMENT LIFE** Anthony Products has validated these coated surgical instruments for twenty-five uses. However, the number of uses obtained from the instruments depends upon the degree of care taken in processing and handling, and the surgical procedures and techniques in which the instruments are used. To achieve maximum life, Anthony Products recommends the following:

- Not allowing gross organic contaminants to dry on the instruments (e.g., blood, mucus, and tissue), by initiating instrument decontamination immediately after completion of the surgical procedure.
- Do not allow instrument tips to touch each other while ESU is active or activated.
- Completely drying instruments before storage.
- Protecting instruments from inadvertent damage while in storage, by wrapping them and avoiding extremes in temperature and humidity.

**NOTE:** Spotting or discoloration may result from inadequate cleaning prior to sterilization or may be due to mineral deposits in water used to autoclave.

**INSPECTION OF INSTRUMENTS** Anthony Products recommends establishing a procedural review, by which the instruments are inspected frequently (before and after

each use) for damages such as:

- Tip misalignment.
- Tip damage e.g., burrs, bending, or discoloration.
- For insulated instruments: cracks, nicks, lacerations, or abrasions in insulation.
- Cracks or nicks in the base of the instrument, where the tines are seated.

Anthony Products recommends establishing a procedural review (before and after each use) by which the instrument showing such damage or wear are either sent for refurbishing, discarded and/or replaced.

**CAUTION:** Using instruments which are damaged or worn can be hazardous to both the patient and operating room personnel.

**REPROCESSING AND STERILIZATION** (i.e., cleaning & sterilization) Institutional device sterilization and reprocessing should occur in facilities that are adequately designed, equipped, monitored, and staffed by trained personnel. Sterilize and clean per your institution's validated procedures and cycle parameters. The following parameters for cleaning and the most commonly utilized methods of sterilization are recommended as guidelines for validation.

**NOTE:** Reprocessing this device dictates that it undergo a thorough cleaning prior to sterilization.

**WARNING:** Clean and sterilize after each use.

#### MANUAL CLEANING

- Rinse instruments thoroughly with sterile purified water to remove any accululated debries.
- Hand wash the surface of instruments using a soft bristled cleaning brush, and enzyme cleaner e.g., Tergazyme™ solution (Alconox, Inc.) or equivalent, to remove visible residual debris. For irrigating instruments, also flush irrigating lumen with approximately 10 ml of enzyme detergent.
- **CAUTION:** Avoid use of abrasive cleaners or solvents.
- After hand washing, the surface is to be thoroughly flushed with sterile, purified water until no visible detergent residual remains.
- For irrigating instruments, also flush irrigating lumen thoroughly with sterile purified water until no visible detergent residual remains.
- Once the instruments are free of cleaning solution and debris, thoroughly dry using a sterile wipe.

**AUTOMATED PRE-CLEANING INSTRUCTIONS** Rinse the instruments under warm running tap water until visibly clean. Use a soft bristle brush (plastic brush) as needed for hard to remove soil. Hard to reach areas such as, internal spaces should be flushed with a water pistol/syringe. Irrigating instruments are provided with a stylet to clean the lumen during rinsing. **CLEANING AND DISINFECTION** Place the instruments in a bath with a tested cleansing and disinfectant agent such as Renu-Klenz™ (Steris) (1/4 oz/gal) prepared according manufacturer's recommendations using lukewarm tap water. The instruments must be completely covered with the solution. **NOTE:** The application times, temperatures, and concentration stated by the manufacturer of the cleansing / disinfectant agent must always be observed. The lumen of irrigating forceps must then be flushed with the prepared detergent. The instruments (particularly irrigating forceps) are then immersed in the detergent solution and allowed to sonicate for ten minutes. Repeat the cleansing process if visible contamination is still present on the instrument. Fresh solutions must be prepared daily. In case of severe soiling, the solution must be changed sooner. A high contamination load in the ultrasonic bath impairs the cleansing action and promotes the risk of corrosion. The cleansing solution must be renewed regularly according to the conditions of use. The criterion is visibly apparent soiling. In any case, a frequent change of bath is necessary, at least once a day. National guidelines must be observed.

**AUTOMATED MACHINE-CLEANING INSTRUCTIONS** The instruments are then to be transferred via a suitable container (e.g., wire mesh basket) into the automated washer. The following cycle is recommended with these parameters programmed; set to high.

Phase	Recirculation Time (minutes)	Water Temperature	Detergent Type and Concentration
Pre-Wash 1	02:00	Cold Tap Water	N/A
Enzyme Wash	02:00	Hot Tap Water	Klenzyme™, 1 oz/gallon
Wash 1	02:00	65oC (set point)	Renu-Klenz™, 1 oz/gallon
Rinse 1	01:00	Hot Tap Water	N/A
Drying	07:00	90oC	N/A

## INSTRUCTION FOR USE (IFU)

### Non-Sterile, Reusable; Electrosurgical Coated Surgical Instruments

The device(s) should then be dried using a clean, soft cloth and visually examined using the naked eye under normal lighting condition to determine that all adherent visible soil (e.g., blood, protein substances and other debris) had been removed from all surfaces, lumens, crevices and serrations. Klenszyme™ and Renu-Klenz™ are trademarks of Steris.

#### STERILIZATION

**NOTE:** Remove stylet if supplied

- **STEAM / GRAVITY DISPLACEMENT: DOUBLE WRAP** instruments in muslin i.e., CSR blue hospital wrap, and place (single layer) in a production type steam sterilization vessel. Process at 132°C (270°F) for 30 minute cycle.
- **STEAM / PRE-VACUUM: DOUBLE WRAP** instruments in muslin i.e., CSR blue hospital wrap, and place (single layer) in a production type, steam sterilization vessel. Process at 132°C (270°F) using pre-vacuum conditions for a 4 minutes cycle
- **CHEMICAL STERILIZATION:** Totally immerse the instruments in CIDEX® activated dialdehyde solution, (Johnson & Johnson Medical, Inc.), or equivalent. Expose the instruments to the CIDEX® for 10 hours at 25°C (77°F). Following chemical exposure, rinse and flush the instruments with copious amounts of sterile water for a minimum of one minute three separate times.
- **STERRAD®100S: DOUBLE WRAP** instruments with Spunguard® Heavy Duty Sterilization Wrap (Kimberly-Clark), or equivalent. Process a total exposure time of 50 minutes diffusion and 15 minutes plasma.

**CAUTION:** Because of the variability of output voltages and modes from generator to generator, **DO NOT USE** electrosurgical forceps with generator setting having a bipolar output voltage exceeding 1200Vp-p. . Refer to the appropriate electrosurgical generator manual for indications and instructions on voltage output characteristics to ensure that all safety precautions are followed. If no RF output is delivered to the accessory handpiece when the generator's activating switch is pressed, check the cord connection with the device and with the generator. If proper function is still not achieved and the accessory handpiece and generator function are confirmed as sound, replace the cord and refer the questionable cord to qualified personnel for further evaluation. **WARNING:** Connect Bipolar accessories to the Bipolar receptacle only, and Monopolar accessories to the Monopolar receptacle. Improper connection of accessories may result in inadvertent accessory activation or other potentially hazardous conditions. Power setting guidelines may vary due to differences in surgical techniques, patients, electrodes and surgical set-up. Start at the lowest power setting and increase as necessary to achieve the desired clinical effect. **PROPER DISPOSAL** of instruments and sharps possibly contaminated with blood, tissue, or other potentially infectious material present a biological risk and must be discarded in a closable, leak-proof, puncture-resistant receptacle, that is adequately labeled (e.g., color coding or symbology) for easy identification as biohazard waste. **RETURNS** Contact your Anthony Products distributor or Anthony Products directly in the event a forceps has to be returned for evaluation. Call Anthony Products Customer Service at 800-428-1610 for a Return Goods Authorization (RGA). When returning forceps follow these instructions:

- **Ship to:**  
Anthony Products, Inc.  
7740 Records Street  
Indianapolis, IN 46226  
800-428-1610  
www.anthonypproducts.com



Anthony Products, Inc.  
7740 Records Street  
Indianapolis, IN 46226  
800-428-1610  
www.anthonypproducts.com



Catalogue Number  
Lot Number  
Fragile, Handle with Care  
See Instruction For Use  
Product is supplied Non-Sterile



Keep dry

Keep away from sunlight

Product Complies with requirements of directive 93/42/EEC for medical devices and harmonized standards BS EN AAMI ANSI IEC 60601-1 & BS EN AAMI ANSI IEC 60601-2-2



**Anthony Products, Inc.**  
7740 Records Street  
Indianapolis, IN 46226  
800-428-1610  
www.anthonypproducts.com