


STERILIZATION GUIDE

Costal Cartilage Graft Cutter

(Item No. AP-WCC)

IMPORTANT: Unit must be fully disassembled before sterilizing.

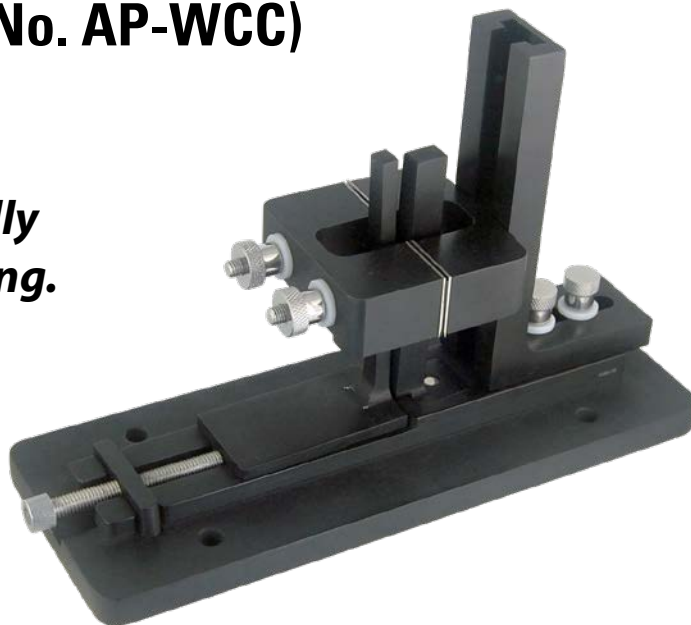

Sterilizer: VAC 08

- **Sterilizer Temperature:** 270.0F
- **Control Temperature:** 273.0F
- **Sterilizer Time:** 5 minutes
- **Dry Time:** 20 minutes

				U=inHg
-	Time	T=F		P=psig
C	12:59:39P.....	271.2.....		29.4P
E	01:04:39P.....	272.6.....		29.8P

Load: 6

06

- **Temp Maximum:** 273.1F
- **Temp Maximum:** 271.2F
- **Condition:** 11:39
- **Sterilize:** 5:00
- **Exhaust:** 20:32
- **Total Cycle:** 37:21

api
anthony products, inc.