

CLEARSCOPE

A D A P T O R

Thank you for choosing the ClearSCOPE Adaptor. We hope you enjoy the product.

Summary & Explanation:

The ClearSCOPE Adaptor enables video recording using a smart phone and endoscope.

Instructions for Use:



1. Ensure your endoscope is in focus. Often there are markings on the endoscope focus ring indicating a neutral position. Ensure your light source is connected and power is on.



2. Connect the magnification lens to the endoscope eyepiece using the c-clamp.



3. Ensure the lens c-clamp is held flush and firm against the endoscope eyepiece, then tighten the locking screw so the lens is stable and secure.



CLEARSCOPE

A D A P T O R



4. Place the edge of a smart phone into the spring-loaded clamps on the adaptor platform and use thumbs to snap into place.



5. Turn over the adaptor platform and use the threaded screw to line up the viewing hole with the smartphone camera lens. Use your thumbs to align the camera lens in the opposite axis.



6. Insert the magnification lens with attached endoscope into the c-clamp on the adaptor platform.




7. Rotate smartphone to desired viewing position and adjust to center image on screen. Use additional locking screw to secure phone and image position.

CLEARSCOPE

A D A P T O R



8. Adjust focus ring on magnification lens or refocus endoscope to perfect image quality.

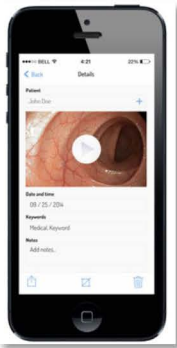

9. Record images and videos using native phone applications, or the ClearSCOPE companion available in the [iTunes app store](#).

An instructional video is also available online [here](#).

Product Registration:

Please register your ClearSCOPE Endoscope Adaptor at: www.clearwaterclinical.com/register/
This will ensure your warranty coverage for your ClearSCOPE and you will receive information and updates about the ClearSCOPE mobile app, which is designed to let you optimize image quality store images and video in a separate, secure library and share images and videos by email or text message.

Warnings and Precautions:

This product is non-sterile. Use caution when manipulating the device so as not to damage your phone or the endoscope.

CLEARSCOPE

A D A P T O R

Cleaning and Disinfecting:

The ClearSCOPE Adaptor is non-sterile. The magnification lens may be cleaned with an alcohol wipe. The magnification lens is not waterproof, although the black plastic mount is submersible. Equipment must be allowed to air dry completely prior to use. Always follow your institutional policies and procedures when cleaning and disinfecting the ClearSCOPE Adaptor or endoscopes.

Patient Privacy:

This product enables image transfer of a potentially sensitive nature. It is the responsibility of the physician/user to respect all rules and regulations that apply in their region / organization, including the 'Health Insurance Portability Accountability Act' or equivalent.

ClearSCOPE Endoscope Adaptor Intended Use:

For enabling image transfer from endoscopes to mobile image capture devices.¹

© 2014 Clearwater Clinical Limited. All rights reserved.
Patents Pending US No. 61/727,465 Assembled in China

For more information, questions or comments please contact
Anthony Products, Inc.
7740 Records Street • Indianapolis, IN 46226
800-428-1610 • (317) 545-6196 • Fax (317) 543-3289
www.anthonyproducts.com