

Sterilization Information

MMT Autoclavable Canisters, Stand and accessories will be shipped as “non-sterile”, reusable device.

NOTE: Canisters and accessories must be soaked immediately after use.

Sterilization Steam:

1. Remove all accessories from canister and ultrasonic clean for 10 minutes using standard medical instrument cleaner.
2. Items should be cleaned thoroughly of all foreign matter, use a brush to clean inside of all parts prior to sterilization.
3. Use a sterile or demineralized lukewarm water to flush all ports. It is recommended to use disinfectant and syringe to flush parts.
4. Canister must be placed separately in a autoclaving bag or sterile wrap. Don't let canisters and accessories make contact directly with metal trays.
5. The maximum sterilization cycle must not exceed 40 minutes at 134°C (273 °F) at 49psi.
6. After removal of the instruments for the autoclave, they should be allowed to cool naturally to room temperature.

Caution: Federal Law (United States) restricts the device to sale by or on the order of a physician or properly licensed practitioner.